

Adhesive Resin Cementation for Indirect Restorations

<p>※2</p>				
<p>※2 Roughen the restorative surface. Rinse and dry.</p>	<p>Apply adhesive. No waiting time.</p>	<p>Gentle air-blow for 3 sec. followed by strong air-blow.</p>	<p>Light cure. LED: 5 sec. Halogen: 10 sec.</p>	<p>Apply resin cement, seat and cure as per manufacturer's instruction.</p>


Hypersensitivity Treatment

<p>Clean treatment site, rinse and dry.</p>	<p>Apply adhesive. No waiting time.</p>	<p>Gentle air-blow for 3 sec. followed by strong air-blow.</p>	<p>Light cure. LED: 5 sec. Halogen: 10 sec.</p>	<p>Remove unpolymerized layer with alcohol swab.</p>

Immediate Dentin Sealing for Indirect Restorations


<p>Prepare cavity or abutment, rinse and dry.</p>	<p>Apply adhesive. No waiting time.</p>	<p>Gentle air-blow for 3 sec. followed by strong air-blow.</p>	<p>Light cure. LED: 5 sec. Halogen: 10 sec.</p>	<p>Remove unpolymerized layer with alcohol swab.</p>

Direct restoration with composite resin

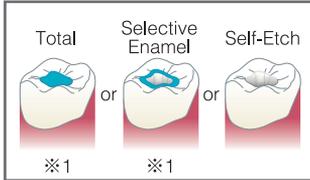

※1 Follow the standard method for acid etch. Rinse and dry.

※2 Rub the adhesive for 20 sec. on phosphoric acid etched tooth surface.

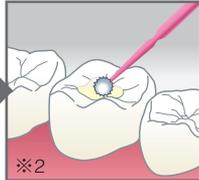

Apply adhesive. No waiting time.



Gentle air-blow for 3 sec. followed by strong air-blow.



Light cure. LED: 5 sec. Halogen: 10 sec.




Restore with composite resin.


Intra Oral Repair




Roughen surface, rinse and dry.

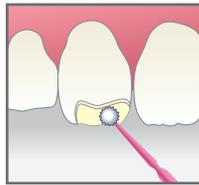


Apply adhesive. No waiting time.

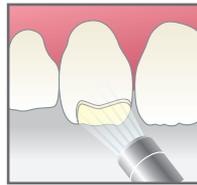

Gentle air-blow for 3 sec. followed by strong air-blow.

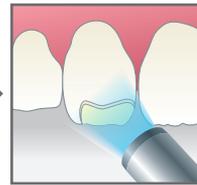


Light cure. LED: 5 sec. Halogen: 10 sec.

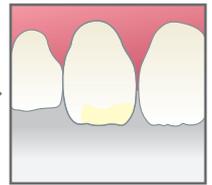


Restore with composite resin.


Core Build-Up

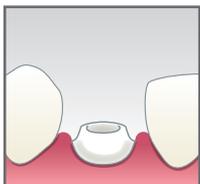


Prepare post space and post as per manufacturer's instruction.

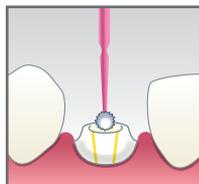


Apply adhesive. No waiting time.

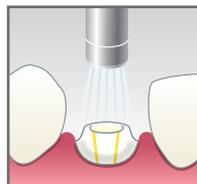

Gentle air-blow for 3 sec. followed by strong air-blow.

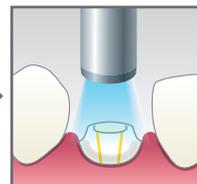

Light cure. LED: 5 sec. Halogen: 10 sec.

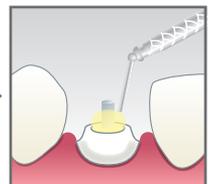

Post cementation and core build-up as per manufacturer's instruction.