

DiaDent® Irrigation Tip End Open Type



Description

This medical device is single use only. Blunt needle tips, luer lock type, non-sterile.

Intended Purpose

Used for irrigation of oral cavity or multi-purpose

- Intended patient population : a patient needed for dental treatment
- Intended user : This medical device shall be used by the qualified doctor.

Instructions for use

Connect the needle tip on the luer lock syringe tightly.

Caution


- 1) For dental use only
- 2) It should be used by a dentist and dental professional. Observe the procedure and the precautions
- 3) Keep it away from a reach of non-dentals including children senior citizens and others.
- 4) Discard the needle with cap after use.
- 5) Do not use after expire date.
- 6) Do not bend the needle tip to avoid needle breakage.

Notice : If any serious incident that has occurred in relation to the medical device, Please report it to the manufacturer and the competent authority.

Storage

- 1) Avoid exposure to direct sunlight and store in a dry cool place.
- 2) Store at room temperature range of 2 ~ 27°C

Symbols

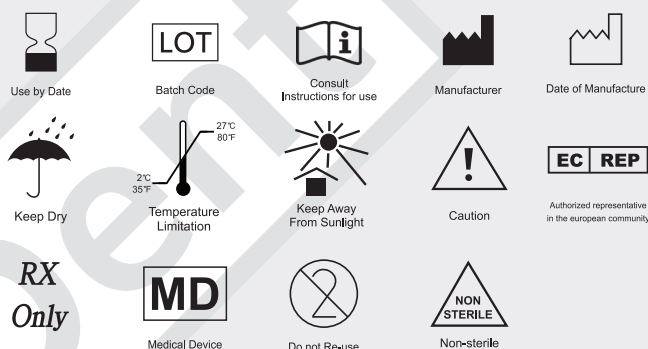

Warranty

DiaDent Group International will replace product that is proven to be defective. DiaDent Group International does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Contact Information

DiaDent Group International

16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu,
Cheongju-si Chungcheong buk-do, 28161, Korea
Tel : 82-43-266-2315
Fax : 82-43-262-8658
www.diadent.co.kr
diadent@diadent.co.kr

DiaDent Group International Inc.

11-3871 North Fraser Way, Burnaby,
B.C., Canada, V5J 5G6
Tel : 1-604-451-8851
Fax : 1-604-451-8865
www.diadent.com
diadent@diadent.com

DiaDent Europe B.V.

Antennestraat 70, 1322AS
Almere, the Netherlands
Tel : 31-36-549-8607
Fax : 31-36-536-7317
www.diadent europe.com
diadent@diadent europe.com



DiaDent Europe B.V.
Antennestraat 70, 1322 AS
Almere, the Netherlands
Tel : 31-36-549-8607