

**Purpose of use**

An absorbent paper points is an endodontic paper points that is used during the endodontic therapy to dry, or remove any foreign materials from the root canal of a tooth.

**Directions for use**

There are two stages to remove the moisture, blood and other foreign substance from the root canal. A dental professional has to follow steps to fill the root canal

1. Before the stage of root canal filling with the Gutta percha points, the absorbent paper points should be checked the length and size to match shape root canal. An absorbent paper points should be selected to be fit to apex of root canal
2. The selected paper points is inserted into the root canal to remove the blood or foreign substance before root canal filling permanently with Gutta Percha Points

This procedure is the most fundamental method. However, a dental professional may consider using more Absorbent Paper Points to perform the procedure in a different way based on the patients symptoms.

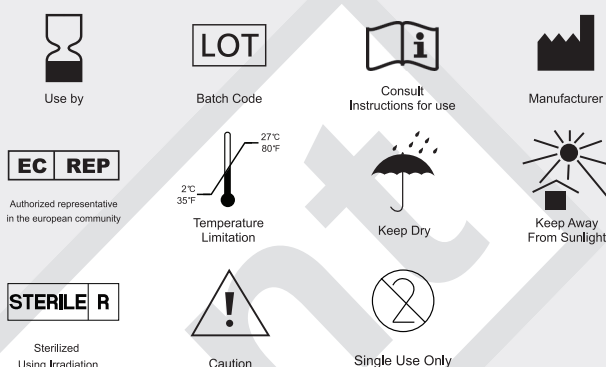
- ◆ Absorbent Paper Points are for a single use.  
Do not re-use them.

**Handling & Storage**

1. Store at room temperature. (2~27°C)
2. Do not throw the products while carrying
3. Store with separate from other products of different sizes and types

**Caution**

1. Sterile Paper Points should be used with the same size and thickness of the Gutta Percha Points.
2. Avoid contact with moisture.
3. Disposable medical products.

**Symbols****Shelf life**

**4 years from date of manufacture**

**Warranty and Limitation of Liability**

DiaDent Group International will replace product that is proven to be defective. DiaDent Group International does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. before using, it is the responsibility of the user to determine the suitability of the product for its intended use. the user assumes all risk and liability in connection therewith.

**EC REP European Representative**

DiaDent Europe B.V.  
Antennestraat 70, 1322AS  
Almere, the Netherlands  
Tel: 00 31 (0)36 549 8607 FAX: 00 31 (0)36 536 7317