

# DiaDent® Steri. Irrigation Tip Side Open Type

CE 0197

## Description

Steri. Irrigation Tip Side Open Type designed as side-port and closed-end type is used for irrigating root canal and periodontal pockets.

## Intended Purpose

This device connected with manually-operated syringe only is used in order to spray root canal with cleansing fluid.

## Instructions for use

- 1) Turn and Fix Steri. Irrigation Tips Side Open Type tightly to the lure lock type syringe.
- 2) Fill the irrigation solution in the syringe, and you use a syringe choose luers that should comply with Steri. Irrigation Tips Side Open Type.
- 2) Check connecting needle and syringe tightly.
- 4) Separate the safe cap from the syringe.

## Caution


- 1) For dental use only
- 2) It should be used by a dentist and dental professional  
Observe the procedure and the precautions
- 3) Keep it away from a reach of non-dentals including children senior citizens and others.
- 4) Discard the needle after use because this product is Sterillized single use.
- 5) Do not re-use the needle which broken the seal regarding as it was used.
- 6) Discard The needle with cap after use.
- 7) Do not use after expire date.

**Notice** : If any serious incident that has occurred in relation to the medical device, Please report it to the manufacturer and the competent authority.

## Storage

- 1) Avoid exposure to direct sunlight and store in a dry cool place.
- 2) Store at room temperature range of 2~27C

## Symbols

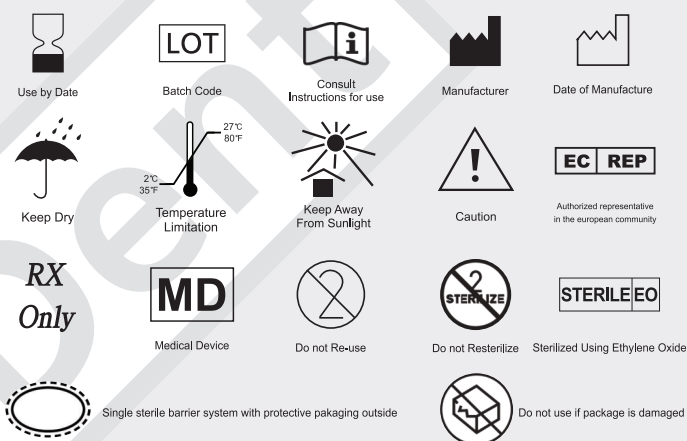

## Warranty

DiaDent Group International will replace product that is proven to be defective. DiaDent Group International does not accept liability for any damage or loss, direct or consequential, stemming from the used of or inability to use the product described. before using, it is the responsibility of the user to determine the suitability of the product for its intended use. the user assumes all risk and liability in connection therewith.

## Contact Information

**DiaDent Group International**  
16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu,  
Cheongju-si Chungcheong buk-do, 28161, Korea  
Tel : 82-43-266-2315  
Fax : 82-43-262-8658  
www.diadent.co.kr  
diadent@diadent.co.kr

**DiaDent Group International Inc.**  
11-3871 North Fraser Way, Burnaby,  
B.C., Canada, V5J 5G6  
Tel : 1-604-451-8851  
Fax : 1-604-451-8865  
www.diadent.com  
diadent@diadent.com

**DiaDent Europe B.V.**  
Antennestraat 70, 1322AS  
Almere, the Netherlands  
Tel : 31-36-549-8607  
Fax : 31-36-536-7317  
www.diadent europe.com  
diadent@diadent europe.com

**EC REP**  
DiaDent Europe B.V.  
Antennestraat 70, 1322 AS  
Almere, the Netherlands  
Tel : 31-36-549-8607